




AB GERMA®

# HEAVY DUTY TRANSFER MATTRESS

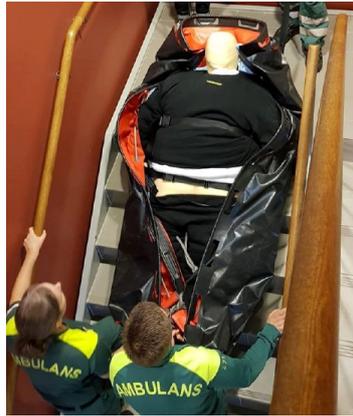

## PRODUCT SHEET

Heavy Duty Transfer Mattress is developed to facilitate heavy duty patient handling. The mattress consists of durable and strong PVC coated polyester fabric material equipped with 6 carry handles on each long side plus evacuation handles on head and foot end of the mattress.

This equipment is specifically designed to support and help and support the paramedics and rescuers to be able to transfer and transport a heavy duty patient. The construction and design make it possible to slide the mattress on the floor or down stair cases with patient onboard the mattress. When needed several rescuers are able lift and carry the patient over obstacles or on to a heavy duty stretcher by using the handles. In next step, from the stretcher to a hospital bed. The purpose is to support handling of heavy duty patient with safety and improved ergonomics.

# HEAVY DUTY TRANSFER MATTRESS



X-RAY MRI AND  
CT TRANSLUCENT



FLAME RETARDANT



HF-WELDED SEAMS

## PRODUCT SPECIFICATIONS

Approved according to European Standard EN 1865, CE-marked and X-ray and MRI translucent

### PRODUCT INFO

Warranty period: 1 year

- 6 carry handles on each side make it easy to find balance for lifting when needed and carry a patient on board. Easy accessible handles up from ground lift.
- 2 handles on head and foot end to slide the patient on floors and staircases
- Sliding surface on the back of the Transfer Mattress.
- Easy to clean fully welded product
- 2 Safety belts to secure the patient and cover sheet to protect and hold the patient
- Durable and strong material and design with a carry capacity of 380 kg.
- Transfer Mattress should be accessible when needed.

### MATERIAL

Material: PVC-polyester fabric material (flame retardant)

Rope: Nylon

Foam: Sensus comfortable memory foam

### SPECIFICATIONS SIZE:

Length: 205 cm

Width: 143 cm

### WEIGHT

8 kg

### ORDER INFORMATION:

Art: Heavy-Duty Transfer Mattress

Art no: 271 050 000



### Visiting address

AB Germa

Industrig. 54-56

SE-291 36 Kristianstad, Sweden

### Postal address

AB Germa

Box 2127

SE-291 02 Kristianstad, Sweden

[www.germa.se](http://www.germa.se)

Phone: +46 (0)44 12 30 30

[info@germa.se](mailto:info@germa.se)

## BENEFITS

Compliant to EN 1865  
Approved for European standard and crash tested for 10-G

One-piece back, providing the patient with even spine support

Also allows preliminary X-ray diagnostics, direct in to x-ray no extra handling of patient.

Germa comfort technology Gives rescuers quick handling and excellent patient ensuring safety is evenly dispersed to give the patient stable and optimal comfort.

Handles ergonomic grip, making patient lifts as comfortable as possible for rescuers

Handles are reinforced so a patient can be lifted in a patient lifting device.

Handles fixed to the bottom of mattress on piece, maintain stable when lifting the patient.

Robust welding—fully welded without any open seams or textiles, making hygienic cleaning of the mattress easy.

X-designed restraint belt fixation close to the patient, support safety and comfort for patient. Supplied with safety straps with quick fix lock, place the straps correctly, easy to stretch and adjust.

Outer material is made of flame retardant PVC-coated fabric It is flexible making it easy to mould the mattress around the patient.

Design of the upper part allow the rescuer to place the arm(s) in different options, make sure to give

# HEAVY DUTY TRANSFER MATTRESS

the rescuer access to patient arm for treatment when requested via patient arm(s)

Extra layer built under the mattress, safety, minimize risk of damage the fixation, last longer.

High load capacity 380 kg, safe to handle heavy duty people.

Closing mechanisms of the straps, plastic buckle Fix lock, allowing the user to quick tighten and adjust restraints on the patient.

PVC coated fabric makes it easy to clean and disinfect with most common disinfectants, easy to keep clean and ready for use.

Repair Kit easy to use for effective repair, quick to repair no return needed.

Visiting address  
AB Germa  
Industrig. 54-56  
SE-291 36 Kristianstad, Sweden

Postal address  
AB Germa  
Box 2127  
SE-291 02 Kristianstad, Sweden

[www.germa.se](http://www.germa.se)  
Phone: +46 (0)44 12 30 30  
[info@germa.se](mailto:info@germa.se)