

BD Eclipse™ with SmartSlip™ technology

The safety needle with unique SmartSlip™ technology helps to ensure that a secure connection is made to a luer slip syringe



Recommended points to practice with luer slip syringe



Drawing up

Use a BD™ Blunt Fill or BD™ Blunt Filter needle to draw up the medication into the syringe. Refer to your local hospital policy for transportation of the filled syringe.

If necessary utilize a passive recapping technique.

Assembly



1. Attach the BD Eclipse™ needle with a firm push straight onto the luer-tip of the syringe. An audible click indicates that the needle and syringe are safely connected.



2. Hold the syringe with the safety cover in an upright position and pull back on the safety cover toward the syringe and away from the needle. The safety cover should be oriented directly above the scale on the syringe.



3. Grasp the syringe with one hand and with the other hand pull the clear needle shield straight off. CAUTION: Do not twist the needle shield when pulling straight off



4. Administer the injection according to your hospital policy
Needle bevel-up position is oriented to the pink safety shield

Activation and Disposal



5 & 6. Activate safety mechanism immediately after injection. Centre your thumb or forefinger on the textured finger pad and push the safety cover forward over the needle until you hear or feel it lock. Visually confirm that the needle is covered.

Use a one handed technique and activate away from self and others. For greatest safety, only use the wide textured finger pad area to activate the safety cover.



7. Discard the BD Eclipse™ safety needle directly into a sharps container. Follow your hospitals local policy for a safe disposal of all medical waste.

