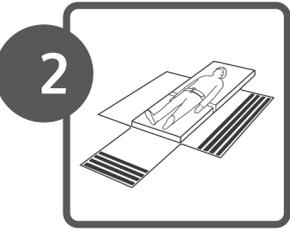
1

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SCAN ME

ENG The fast and safe way to evacuate immobile people. Can be easily dragged by just one rescuer.

Safety information

These instructions contain information concerning the safe use of the S-CAPEPLUS. To ensure that the S-CAPEPLUS is used correctly and safely, this information must be read prior to placement and usage of the S-CAPEPLUS.

- Always observe the instructions given in the instruction manual when using the S-CAPEPLUS.
- Only use the S-CAPEPLUS in emergencies, such as a fire, power cut, gas leak, terrorist attack, natural disaster, extreme smoke or any other life-threatening situation or emergency.
- Product training with the S-CAPEPLUS must always be given by a certified emergency response team member.
- The use of alcohol and/or prescription and over-the-counter medicine may have a negative effect on the skill of the rescuer who pulls the S-CAPEPLUS.
- Do not remove any parts of the S-CAPEPLUS.
- Make sure the safety information and instruction manual are kept close to where the S-CAPEPLUS has been installed.
- This S-CAPEPLUS is not intended for recreational use.
- Do not use flammable vapours, materials or liquids, such as petrol, alcohol, etc., close to the S-CAPEPLUS.
- Not observing the instructions given in the instruction manual provided with the S-CAPEPLUS may result in

serious physical injury or even death.

- Sharp edges or protrusions may damage the S-CAPEPLUS and, therefore, reduce the effectiveness of the product.
- Practice using the S-CAPEPLUS at least once a year in order to guarantee its correct and safe use in the event of an emergency.
- Contact the various rescue services (fire brigade, ambulance service, etc.) to inform them of the purchase and presence of the S-CAPEPLUS.
- The S-CAPEPLUS is not suitable to be pulled over a naked flame.
- The S-CAPEPLUS must be carefully checked for damage after it has been used. If damage is observed, the S-CAPEPLUS must not be reused. Furthermore, the S-CAPEPLUS must be folded up and stored away each time after use.
- The S-CAPEPLUS has a 1-year warranty for material and manufacture faults, which is valid from the date of purchase. The manufacturer may ask you for proof of the purchase date. Therefore, it is recommended to keep the invoice in a safe place. Faults or problems with the use of the S-CAPEPLUS as a result of an accident, misuse, incorrect use, alterations, vandalism, incorrect placement, incorrect maintenance or administration, as well as overdue normal and routine maintenance, are not covered by the warranty.
- The manufacturer is not liable for (consequential) damage as a result of the use of the S-CAPEPLUS.

Never overestimate yourself! Evacuation via a staircase can be very dangerous! You will have to support a heavy weight and you must be certain that you are capable and strong enough to keep this weight under control all the way down the staircase. If in doubt as to whether you are able to evacuate somebody down a staircase yourself, you must always ask a second care worker to help you or leave the task to someone else. Your own safety is of the utmost importance! Regularly practicing using the S-CAPEPLUS during evacuation exercises and always observe the instructions given below.

Only use in case of emergency situations. The manufacturer and supplier cannot be held liable for any improper use!

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