
ri-thermo® tymPR0+

Quick Start Guide



ORDERING INFORMATION

No. 1835-BT	Tympanic thermometer ri-thermo® tymPRO+ Bluetooth
No. 1835	Tympanic thermometer ri-thermo® tymPR0
No. 12634	Probe cover for ri-thermo® tymPR0 1 rack / 10 pcs
No. 12640	Probe cover for ri-thermo® tymPR0 30 racks / 300 pcs
No. 12911	Anti-theft cable, length 50 cm, black, keyring at both end for ri- thermo® tymPR0
No. 12953	Calibration kit for ri-thermo ® tymPR0
No. 13347	Bluetooth connectivity: Bluetooth USB-Dongle for RVS- 100

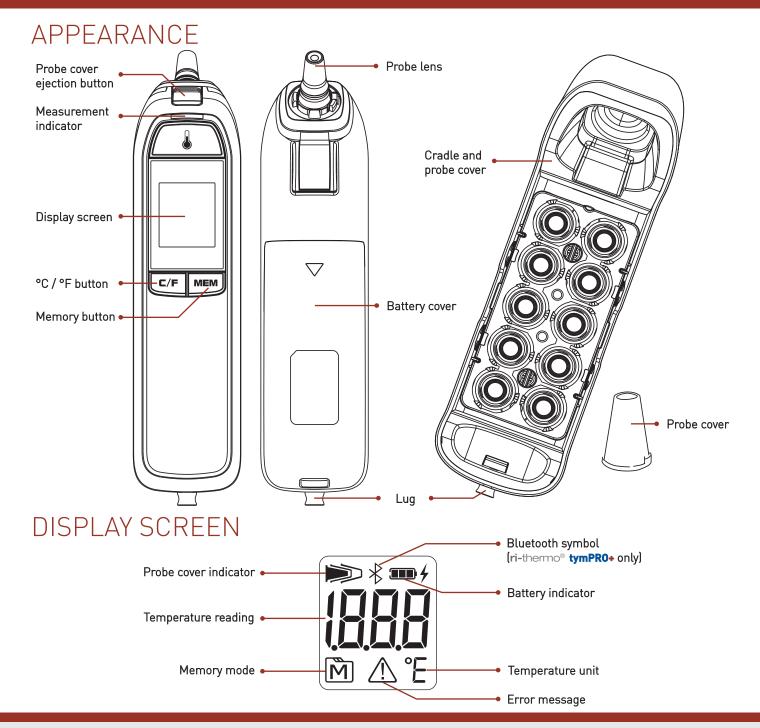


USING THE DEVICE

Measuring Temperature






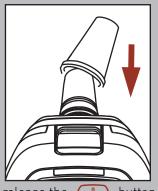


STEP 1. To turn on **ri**-thermo® **tymPRO**, briefly press and release the buttor ", when flashing, this indicates the **tymPRO** is ready to use and a new probe cover should be attached. Alternatively, attach a new probe cover to turn it on. When ready, the thermometer displays the last measurement.

STEP 2. Gently fit the probe with a new probe cover into the ear canal. Ensure that the probe shaft is aligned with the ear canal.

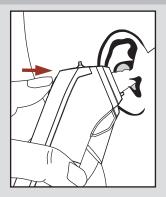



STEP 3. Briefly press and release the button.
You will hear a beep to indicate measurement is complete and thermometer may be removed. The indicator blinks during the measurement and turns

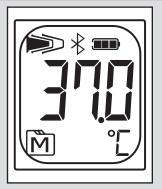


STEP 4. Read the result. The Bluetooth symbol is for **ri**-thermo® **tymPRO**+ only.




STEP 5. Press ejection button to discard used probe cover into clinical waste bin or recycling facitilies (in line with local guidelines).

Replacing Probe Cover

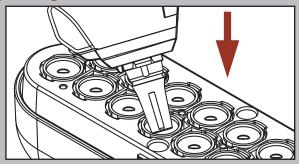

solid blue when the measurement is completed.

STEP 1. Firmly insert the thermometer probe into new probe cover in the probe cover cassette. You will hear two clicks when the probe cover is attached properly.

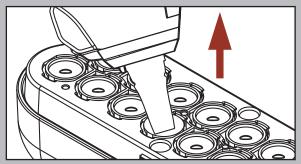

STEP 2. Lift the probe out of the cassette with a new probe cover attached. Do not touch the cover. Now, you can measure the temperature.

Please visit riester.de for more information



Refer to instructions for use for full functional usage